Blephasteam®

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EN

Blephasteam®

Eye-warming device, moist heat technology


BLEPHASTEAM® is a patented medical device based on the results of scientific research by Dr J R Fuller. This innovative device reproduces an environment that naturally enhances tear film quality and stability by reinforcing the thickness of the tear film lipid layer.

This layer prevents tear film evaporation; it is produced by Meibomian glands which are found in the eyelids. These glands can easily become blocked, causing conditions such as dry eye syndrome, chalazion, blepharitis or ocular rosacea.

BLEPHASTEAM® spreads latent heat therapy which allows the melting of secretions and unblocks the Meibomian glands, improving tear film stability. Thanks to this action, **BLEPHASTEAM®** leads to improved eyelid health and daily comfort.

BLEPHASTEAM® has been tested and validated to ensure safe use.

While using **BLEPHASTEAM®**, vision remains clear and the eyelids function normally, which promotes secretion of the lipid tear film and increased tears during blinking. The temperature reached at cornea level is 42 ± 1 °C; this temperature remains constant during use. The component parts of the device are permanently connected. The device is intended for use at home.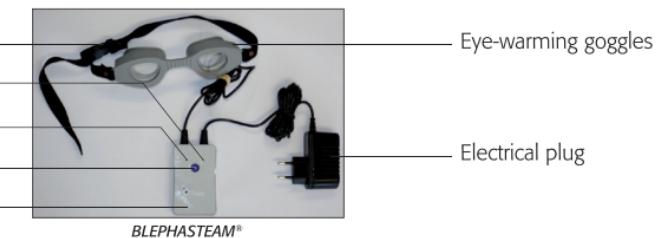

Important:

BLEPHASTEAM® should be plugged in for 15 minutes before use to allow the device to preheat (see instructions for use in paragraph VIII).

The orange and green lights on the control box are to indicate:

- Orange The device is preheating.
- Green continuous Preheating is over and the device is ready for the treatment session.
- Green flashing The device is in use (treatment in progress) and the timer is on for 10 minutes.

II. INDICATIONS

BLEPHASTEAM® relieves the symptoms caused by abnormal functioning of Meibomian glands. This abnormal functioning is associated with a number of eye conditions such as dry eye syndrome, chalazion, blepharitis or ocular rosacea.

The heat and moisture provided by the device melt the secretions that block these glands, making it easier to remove the secretions using massage and pressure of the eyelids.

For any questions about this device or for medical advice, please contact your eye specialist or Laboratoires THEA (see contact details at the end of this leaflet).

III. WARNINGS

BLEPHASTEAM® should be used by children only when supervised by an adult and after talking to an eye care specialist.

If you wear contact lenses or prescription glasses, always remove them before using BLEPHASTEAM®.

If you have a reaction to BLEPHASTEAM®, such as excessive sensitivity to heat or an allergic reaction to one of the components of the device, stop the treatment immediately and talk to your doctor.

Eye drops should not be used for at least 15 minutes before using BLEPHASTEAM®.

IV. SIDE EFFECTS

The following side effects have been reported:

- Eye/eyelid irritation and redness
- Eye pain.

If you notice any side effects or unusual sensations after using this product, please contact your doctor, pharmacist or nurse and report the information to the local distributor or to the manufacturer (see contacts at the end of these instructions for use).

V. CONTRAINDICATIONS

Unless otherwise specified by your doctor, BLEPHASTEAM® must not be used in the following cases:

- Acute diseases of the eyes and/or eyelids,
- Recent injury and corneal lesions or damage to the eye,
- Meibomian seborrhoea (excessive secretion of sebum produced by the Meibomian glands),
- After recent surgery to or around the eye,
- If you are allergic to any component (plastic component).

VI. PRECAUTIONS FOR USE AND WARNINGS

Read carefully before use

This medical device must be used in accordance with these instructions. If not, the safety and functioning of the equipment are put at risk and may damage the device.

PRECAUTIONS FOR USE

- This device should not be used by any patient (including children from 3 years old) who has low physical, sensory or mental abilities unless supervised and on the advice of an eye care specialist.
- This is a medical device that should only be used by a capable adult (patient) or, when used by children (from 3 years old) or certain adults (patients), under the supervision of a capable adult.
- Keep the device out of the reach of children.
- BLEPHASTEAM® can be used in children from 3 years old. The device must be used in the presence of an adult.
- If you have a reaction to BLEPHASTEAM®, such as excessive sensitivity to heat or an allergic reaction to one of the components of the device, stop the treatment immediately and talk to your doctor.
- The BLEPHASTEAM® power plug must always be accessible in order to disconnect the system in case of issue. This instructions manual should be available to the user at all times and the user should read all instructions carefully before using the device.

WARNINGS

- Do not put or use this device in places where it may come into contact with a liquid or be exposed to environmental conditions such as very high temperatures, damp, direct exposure to sunlight, dust, salt, etc., which could damage it.
- You do not need to perform any regular maintenance on this device. When maintenance is required, the device will not pass its function check.. When the function check is not successful, the user is prompted to contact technical support or the manufacturer (see § XII).

- Never attempt to access the internal hardware for any purpose including maintenance.
- Do not install or use this device in the presence of gas or inflammable or explosive chemicals.
- When in use, do not block access to the electricity supply plug in case you need to unplug it quickly.
- Do not disassemble, modify or repair the instrument by yourself. Otherwise, it may cause fire, electrical shock, bodily injury, or instrument malfunction. Refer all servicing to Laboratoires THEA or your authorised THEA dealer. The instrument disassembled, modified or repaired by anyone other than a Laboratoires THEA designated repair facility will void the warranty.
- Remove the plug from the socket by pulling on the plug and not on the lead as this may damage the plug.
- Do not expose the device or rings to very high temperatures or flames or sunlight.
- Do not immerse the device in water or clean it in running water.
- Do not clean with any chemical products or solvents.
- **BLEPHASTEAM®** is supplied with a power plug cable. Do not use any other cable, it could negatively affect electromagnetic compatibility performance of the device. Use only the cable supplied with the device to connect the device to the electrical plug.
- Do not use the device if it is damaged. Check all the cable surfaces for any defects; cables must be intact.
- If the electrical cable is damaged, it must be replaced by the manufacturer, its after-sales service or similarly qualified persons to avoid any danger.
- Do not bend, crush or excessively strain the cords or cables.
- Cables can cause strangulation. Do not attach or tighten cables to or around the head or neck.
- Do not connect the device up to another appliance or to another source of power and check that the mains power socket is not faulty or damaged.
- Check that the voltage shown on the electrical plug of the device is the same as the voltage of your power source (using the wrong voltage may cause lasting damage that is not covered by the warranty).
- Do not use an extension lead.
- The use of accessories other than those specified for the device is not recommended. They may result in increased emissions or decreased immunity of the device.
- The use of accessories other than those specified for the device is potentially dangerous. Never connect an accessory which is not listed in these instructions for use.
- It is forbidden to tamper with the device in any way. Do not dismantle or reassemble the device.
- The device must not be stacked and located close to other equipment.
- **BLEPHASTEAM®** goggles have to be used with **Moisten BLEPHASTEAM® rings** (**See paragraph VII**) for a temperature of $42 \pm 1^\circ\text{C}$ reached at cornea level. Without Moisten rings, the temperature of the glasses can reach 66°C .
- **Small Children:** Do not leave your Blephasteam and its accessories within the reach of small children or allow them to play with it. They could hurt themselves or others, or could accidentally damage the **BLEPHASTEAM**. Your **BLEPHASTEAM** contains small parts with sharp edges that may cause an injury or may become detached and create a

choking hazard.

- **Animals or insects:** Do not leave your Blephasteam and its accessories within the reach of animals or insects. After treatment, your device must then be put away in its storage bag.

VII. RECOMMENDATIONS FOR USE

It is recommended that you use **BLEPHASTEAM®** twice a day, unless your eye specialist tells you otherwise. Allow at least four hours before each treatment session.

It is important that the instructions for use (in paragraph VIII) and the recommendations given by your eye specialist are followed.

Your **BLEPHASTEAM®** device is designed to be used by one person only and must be cleaned after each use (see paragraph IX).

It is compatible with a power voltage supply between 100-240 Volts/50Hz-60Hz/160-80mA.

The device must be used with previously moistened **BLEPHASTEAM®** rings only. These rings are for single use only and must be thrown away after each use to guarantee maximum efficacy of the treatment.

The **BLEPHASTEAM®** rings are not supplied with the **BLEPHASTEAM®** device and must be ordered from Laboratoires THEA or from www.blephasteam.com

Keep your **BLEPHASTEAM®** device stored in the bag supplied between each treatment session.

Use this device at room temperature, between 5°C and 40°C .

During storage and transportation, the device must not be subject to temperatures below -10°C or above 40°C .

For further information, please contact your eye specialist or Laboratoires THEA.

VIII. INSTRUCTIONS FOR USING BLEPHASTEAM® DEVICE

Wash your hands before and after use.

Use on clean eyes without any make-up. Contact lenses and prescription glasses must be removed before the treatment session.

To be more effective and so that the rings remain in place, **BLEPHASTEAM®** should not be used lying down.


It is best to be seated and not to move when using the device.

Blink normally while using the device.

STEPS 1 to 3: Preheating
 STEPS 4 to 9: Treatment
 STEPS 10 to 11: Cleaning/Putting away
 Follow all the steps to carry out a treatment session.


PREHEATING:

STEP 1
 Connect the plug up to the mains socket
 and leave it plugged in throughout the session.
 You should hear a beep.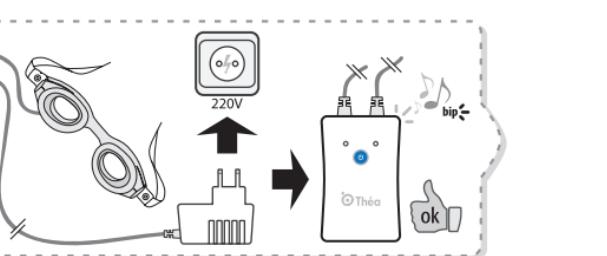

STEP 2
 Press the On button (middle blue button) firmly:
 preheating begins; the orange light comes on.

NOTE: The On button allows you to start the preheating stage.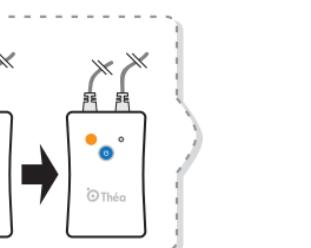


STEP 3
 After 15 minutes, the green light comes on: preheating is over.
 The device is now ready for use and the treatment session can begin.

You have 45 minutes to start your session.
 Once this time is up, the device goes off and has to be restarted (start again at step 1).

TREATMENT:

STEP 4
 Moisten two BLEPHASTEAM® rings, preferably using mineral water
 or physiological salt solution (do not soak the rings or they will drip).
 Keep away from the plug during this step so you do not wet it.

STEP 5
 Place these moist rings inside the device, one ring per lens.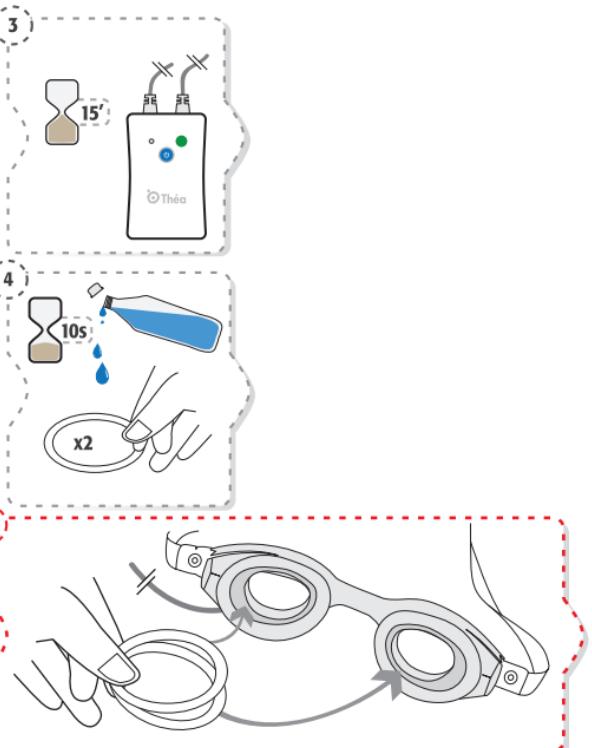

STEP 6

Fit the device comfortably over your eyes and **adjust them** to fit your head by using the strap. The rings must remain in place in line with the goggle lenses.

Note: Temperature of the outer goggle can reach 46°C around the glasses.
Use the strap for adjustment and limit the contact time.
With the outer goggle to 10 sec to avoid any discomfort.

STEP 7

To start the session, **press the On button (middle blue button)**. The green light should now start to flash. The treatment session lasts 10 minutes. Remain seated during the session. Do not unplug the device during the treatment.

NOTE:  The On button allows you to start the treatment session.

STEP 8

The session is over after 10 minutes.
The device stops automatically: you will hear a beep and the green light goes off.

**STEP 9**

Remove the device, disconnect the plug from the socket and throw the rings away in the bin.

CLEANING / PUTTING AWAY:**STEP 10**


Massage your eyelids, and then clean them with a compress such as **BLEPHACLEAN®**.

STEP 11

Clean the device with the blue cloth provided or an alcohol-soaked compress, taking care to wipe the whole device and the lenses.

Put the device and the blue cloth away in the storage bag.

If the session is cut short for any reason, unplug the device and start again at Step 1.

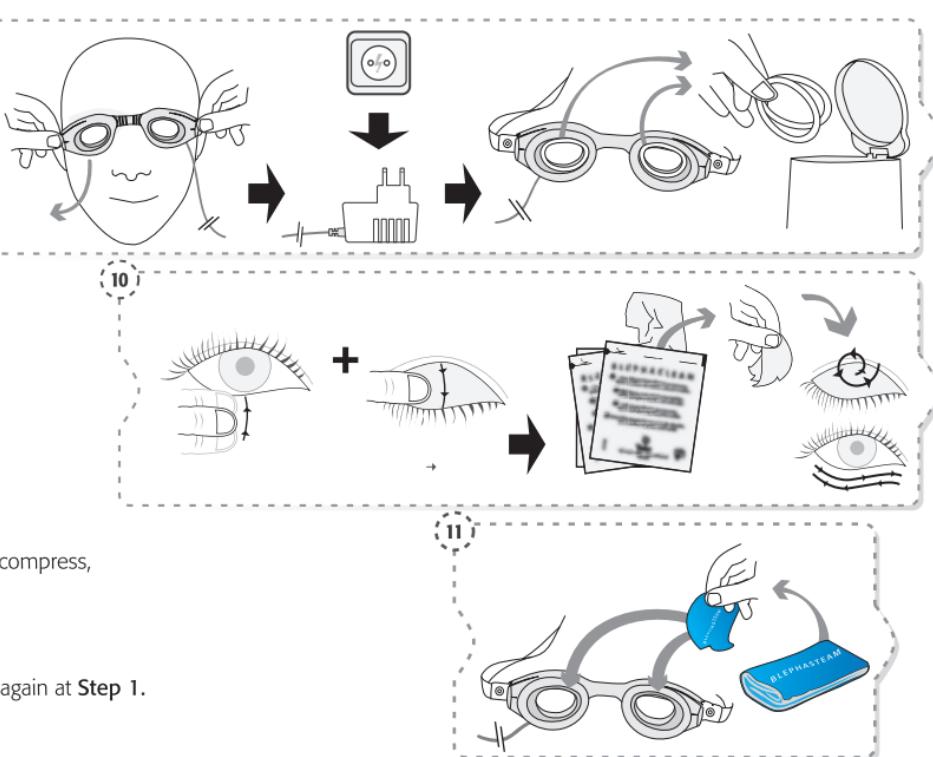IX. CLEANING AND PUTTING AWAY YOUR BLEPHASTEAM® DEVICE

Regularly clean dust off the device using a soft cloth. Use the blue cloth provided or a soft cloth moistened with 70% alcohol solution for persistent dirt on the surface. Use a lens wipe to clean the glass protecting the lens. Gently wipe and take care not to scratch the glass. The device must be disconnected from the power supply when you clean it.

The device must then be put away in its storage bag.

X. WARRANTY AND SERVICES

The warranty is valid ONE YEAR from the date of purchase. The warranty covers any fault, material damage or manufacturing defect in products used in accordance with the instructions in this leaflet.

This warranty does not preclude the application of current legal guarantees under national legislation governing the sale of consumer goods.

The warranty does not apply in the following cases:

- Damage during transport,
- Improper use or negligence,
- Incorrect handling,
- Modification of the system,
- Poor maintenance,
- Use of wrong voltage,
- Lightning,
- Infiltration of sand or water.

XI. MAINTENANCE AND INSTRUCTIONS IN CASE OF BREAKDOWN

Preventive inspection

Check that your **BLEPHASTEAM®** device is working properly at least once a year and if the device is not used for a long time (more than a month). To do that, follow these steps:

- Carry out steps 1 to 3 of paragraph VIII. INSTRUCTIONS FOR USE.
- Time the preheating period (for the whole time the orange light is on).
- Record the preheating time in the table below, in the "Preheating time" column. The time measured should be between 14 and 16 minutes.
- Check that the goggles are producing heat.
- Carry out step 7 of paragraph VIII. INSTRUCTIONS FOR USE to start a 10-minute session.

- Time the length of the treatment session (for the whole time the green light is flashing).

- Record the treatment session time in the table below, in the "Heating time" column. The time measured should be between 9 and 11 minutes.

If one of the 2 times does not come within the duration given OR if the device does not produce any heat, then it is no longer operational. In that case, do not use the device and return the product to Laboratoires THEA in its original packaging, with this leaflet and with the sales receipt and the reason for returning the device.

Date	Preheating time (Orange light) 14-16 min	Heating time (Green flashing light) 9-11 min

The device does not require any maintenance or calibration.

The device has a life time of 7 years.

In the event of a fault or malfunction, please return the product to Laboratoires THEA in its original packaging and enclose the sales receipt and the reason for returning the device.

For any other problem or question about how this system works, please contact Laboratoires THEA.

XII. MANUFACTURER AND EUROPEAN DISTRIBUTOR

Laboratoires THEA
12, rue Louis Blériot
63017 Clermont-Ferrand
CEDEX 2 - FRANCE

XIII. TECHNICAL INFORMATION

The BLEPHASTEAM® device operates with a power supply voltage between 100-240 Volts/50Hz-60Hz/160-80mA.

Reference for the BLEPHASTEAM® device: T2420. Supplied with a storage bag and a blue cleaning cloth.

The BLEPHASTEAM® rings are not supplied with BLEPHASTEAM® and must be ordered from Laboratoires THEA or from www.blephasteam.com

Control box / electric cable:

Power supply

Reference: FW8002M/09 (article number: 1960031).

The minimum voltage is 9V and the current is 0.8A.

Plug adapter is the isolation element of the network

Disposing of this electrical device

According to Directives 2012/19/UE WEEE and 2011/65/UE RoHS II on the restriction of hazardous substances in electrical and electronic equipment on their disposal.

Public authorities adopt adequate measures to make sure that users, distributors and manufacturers contribute to the collection of electrical and electronic equipment, setting legal requirements for reusing, recovering or recycling said equipment.

The user must take into account the potentially harmful effects to the environment or human health due to the improper disposal of the equipment or of parts of it.

This graphic symbol shown in the figure is applied on the equipment's label.

It reminds that all electrical and electronic equipment must be collected and disposed of separately at their end-of life.

Your BLEPHASTEAM® device shall be handed over to a waste collection centre or taken back to Laboratoires THEA if another device is purchased.

Symbols used and their definition

	Applied part: Type BF (safety level specified by standard IEC 60601-1)		Batch number
	Do Not Reuse (rings)		Product reference
	Home use		Expiry date
	Refer to the instruction leaflet		Do not dispose of with household waste
	Date of manufacture		Manufacturer
	Temperature limits within which the device can be safely exposed		Humidity limits within which the device can be safely exposed
	Pressure limits within with the device can be safely exposed	IP 22	Ingress Protection indice : Protects persons against access to hazardous parts with fingers and against the harmful effects due to direct sprays of water up to 15° from the vertical

Ambient Conditions

Transport	Temperature -10°C to +40°C (14°F to 104°F) Atmospheric pressure 700 hPa to 1060 hPa Relative humidity 15% to 93%
Storage	Temperature -10°C to +40°C (14°F to 104°F) Atmospheric pressure 700 hPa to 1060 hPa Relative humidity 15% to 93%
Operating	Temperature +5°C to +40°C (41°F to 104°F) Atmospheric pressure 700 hPa to 1060 hPa Relative humidity 15% to 93%

ELECTROMAGNETIC COMPATIBILITY**GENERAL PRECAUTIONS AND WARNINGS**

- Electrical medical devices and systems are subject to special measures concerning electromagnetic compatibility (EMC) and must be installed in accordance with the EMC instructions contained in this manual.
- Portable and mobile radiofrequency communication systems may interfere with electrical medical devices. The **BLEPHASTEAM®** is intended for use in an electromagnetic environment where radiated RF disturbances are under control. The customer or the user of the device can help prevent electromagnetic interference by maintaining a minimum distance (30 cm or 12 inches) between mobile and portable RF communication devices (transmitters) and the **BLEPHASTEAM** device, according to the maximum output power of the radio communications devices.
- Use of accessories and cables other than those supplied with the instruments, except the cable sold by the equipment manufacturer as spare parts, may lead to an increase in emissions and reduce the device's or system's immunity.
- The device must not be used in contact with other devices.
- This instrument is not designed to be used next to or on top of another instrument. Therefore, do not use it like that use. Nevertheless, if such use is inevitable, it is necessary to monitor constantly to ensure the instrument is functioning normally.

Precautions

ELECTROMAGNETIC EMISSIONS

The device is intended for use in the following electromagnetic environment. The user must ensure compliance with this guideline.

TABLE 7 : ELECTROMAGNETIC EMISSIONS

Guidance and manufacturer's declaration – electromagnetic emissions		
The BLEPHASTEAM® is intended for use in the electromagnetic environment specified below. The operator of the BLEPHASTEAM® has to make sure that it is used in such an environment.		
Emission test	Compliance	Electromagnetic environment - Guidance
Radiated RF emission acc. to CISPR 11	Group 1 Class B	The BLEPHASTEAM® uses RF energy only for its internal functions. Therefore, its RF emissions are very low and are not likely to impair nearby electronic equipment.
Conducted RF emissions acc. to CISPR 11	Group 1 Class B	The BLEPHASTEAM® is suitable in all establishments
Harmonic emissions acc. to IEC 61000-3-2	Compliant	
Voltage fluctuations / Flicker emissions acc. to IEC 61000-3-3	Compliant	

ELECTROMAGNETIC IMMUNITY

The device is intended for use in the following electromagnetic environment. The user must ensure compliance with this guideline.

TABLE 8: GUIDANCE ON MANUFACTURER DECLARATION – ELECTROMAGNETIC IMMUNITY

Immunity test	Compliance level	Notes
Electrostatic discharge (ESD) acc. To IEC 61000-4-2	± 8 kV contact ± 2 kV, ± 4 kV, ± 8 kV, ± 15 kV air	The device is intended for use at home

Radiated RF EM fields acc. To IEC 61000-4-3	10 V/m 80 MHz – 2.7 GHz 80% AM at 1 kHz	The device is intended for use at home
Proximity fields from RF wireless communications equipment acc. To IEC 61000-4-3	See table 9	The device is intended for use at home
Electrical fast transients/burst acc. To IEC 61000-4-4	- ±2 kV/100 Hz for power supply lines	The device is intended for use at home
Surge acc. To IEC 61000-4-5	- ±0.5, ±1 kV differential mode	The device is intended for use at home
Conducted disturbances induced by RF fields acc. To IEC 61000-4-6	3 V 0.15 MHz – 80 MHz 6 V in ISM and amateur radio bands between 0.15 MHz and 80 MHz 80% AM at 1 kHz	The ISM (industrial, scientific and medical) bands between 0.15 MHz and 80 MHz are 6.765 MHz to 6.795 MHz; 13.553 MHz to 13.567 MHz; 26.957 MHz to 27.283 MHz; and 40.66 MHz to 40.70 MHz. The amateur radio bands between 0.15 MHz and 80 MHz are 1.8 MHz to 2.0 MHz, 3.5 MHz to 4.0 MHz, 5.3 MHz to 5.4 MHz, 7 MHz to 7.3 MHz, 10.1 MHz to 10.15 MHz, 14 MHz to 14.2 MHz, 18.07 MHz to 18.17 MHz, 21.0 MHz to 21.4 MHz, 24.89 MHz to 24.99 MHz, 28.0 MHz to 29.7 MHz and 50.0 MHz to 54.0 MHz.
Voltage dips, short-term interruptions and voltage variations on power supply input lines acc. to 61000-4-11	0% UT; 0.5 cycle At 0°, 45°, 90°, 135°, 180°, 225°, 270° and 315° 0% UT; 1 cycle and 70% UT; 25/30 cycles Single phase: at 0° 0% UT; 250/300 cycle	The device is intended for use at home
Power frequency (50/60 Hz) magnetic fields acc. to IEC 61000-4-8	30 A/m 50 Hz or 60 Hz	The device is intended for use at home

TABLE 9: TEST SPECIFICATIONS FOR ENCLOSURE PORT IMMUNITY TO RF WIRELESS COMMUNICATIONS EQUIPMENT

Test frequency (Mhz)	Band ^{a)} (Mhz)	Service ^{a)}	Modulation ^{b)}	Maximum power (W)	Distance (m)	IMMUNITY TEST LEVEL (V/m)
385	380-390	TETRA 400	Pulse modulation ^{b)} 18 Hz	1.8	0.3	27
450	430-470	GMRS 460, FRS 460	FM ^{c)} ± 5 kHz deviation 1 kHz sine	2	0.3	28
710						
745	704-787	LTE Band 13, 17	Pulse modulation ^{b)} 217 Hz	0.2	0.3	9
780						
810						
870	800-960	GSM 800/900, TETRA 800, iDEN 820, CDMA 850, LTE Band 5	Pulse modulation ^{b)} 18 Hz	2	0.3	28
930						
1720						
1845	1700-1990	GSM 1800; CDMA 1900; GSM 1900; DECT; LTE Band 1, 3 4, 25; UMTS	Pulse modulation ^{b)} 217 Hz	2	0.3	28
1970						

2450	2400-2570	Bluetooth WLAN 802.11 b/g/n, RFID 2450, LTE Band 7	Pulse modulation ^{b)} 217 Hz	2	0.3	28
5240						
5500	5100-5800	WLAN 802.11 a/n	Pulse modulation ^{b)} 217 Hz	0.2	0.3	9
5785						
NOTE If necessary to achieve the IMMUNITY TEST LEVEL, the distance between the transmitting antenna and the ME EQUIPMENT OR ME SYSTEM may be reduced to 1 m. The 1 m test distance is permitted by IEC 61000-4-3						
a) For some services, only the uplink frequencies are included. b) The carrier shall be modulated using a 50% duty cycle square wave signal. c) As an alternative to FM modulation, 50% pulse modulation at 18 Hz may be used because while it does not represent actual modulation, it would be worst case.						

Date of instruction leaflet upgrade: January 2021

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Théa Pharma GmbH
Thea Pharma GmbH
Thea Farma S.p.A.